SCORPION

Instruments dentaires

Notice d'utilisation
Instruction for use
Istruzioni per l'uso
Instrucciones de empleo
Bedienungsanleitung

Insert de détartrage dentaire
Dental scaler tip
Inserti per detartraggio
Insert de detartraje
Scaler spitze

Scorpion

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Indications: The Scorpion scaler tip is perfectly designed for teeth scaling. Due to an improved vibration, power on the scaler equipment can be reduced by the dentist from 20 up to 40 % in comparison with a classical tip. This offers a better comfort for the patient with an unchanged efficiency for the dentist. Leave the insert remove the tartar without applying pressure. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour.

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Indications: The EP Scorpion scaler tip is perfectly designed for teeth scaling in **Periodontology**. Its special long end will provide the perfect vibration during deep scaling within perio pockets. Set your scaler unit to **Periodontology**. Higher ultrasonic settings may fracture the tip. Use the EPSA Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pockets irrigation during treatment to avoid an excessive heating.

Sterilization: Please follow up to date procedures with autoclaves or dry heat sterilizer.

Note: The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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Indications: The Scorpion AIR scaler tip is perfectly designed for teeth scaling with the SYNEA scalers of W&H company and the AIR scalers of NSK company. For a better patient comfort, parasite vibrations are totally eliminated. With W&H scalers, adjust your handpiece power to Stage 1. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour.

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