**EN** **FR** **DE** **IT** **ES**

# SEcure

Self-etching Dual-cure Resin Cement

## EN

**INSTRUCTIONS**

**IMPORTANT :**
**READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.**
**KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.**

Self-etching Dual-cure Resin Cement

## SEcure

**FOR DENTIST USE ONLY**

#### 1. What is SEcure?

SEcure is a self-etching, dual-cure resin cement for onlays, inlays, crowns and bridges. It contains 4-META\*, a high performance adhesive monomer that creates excellent bonds to dentin and enamel by virtue of formation of a hybrid layer. Chemically SEcure is a resin cement and thus water resistant. Removal of excess resin is quick and easy. The cement is radiopaque.

The pretreatment with V-PRIMER for precious metals and Porcelain Liner M for porcelain will further enhance the bond strength and its durability.

\* 4-methacryloxyethyl trimellitate anhydride

#### 2. Contents of the kit

①Dual-cure Resin Cement (in double-barrel syringe) ..... 9g (Shade : Translucent)
②Primer ..... 3mL
③Mixing Tip ..... 10 pieces

**Sold separately :**

- ①Dual-cure Resin Cement (in double-barrel syringe) ..... 9g (Shade : Translucent, White Opaque, Dentin)
  - ②Mixing Tip ..... 10 pieces
- Primer ..... 3mL
- Mixing Tip ..... 30 pieces

#### 3. Precautions : Read all instructions thoroughly before use.

##### 3.1 Safety

- ①**Sensitivity**  
SEcure should not be used by clinicians or on patients who are sensitive to methacrylates or resin cements. If signs of irritation such as redness appear, stop using SEcure immediately and consult a physician.
- ②**Avoid contact**  
Avoid contacting soft tissues, skin or eyes by SEcure components. The use of rubber dam is recommended. Contaminated skin or mucosa should be wiped immediately with alcohol and then rinsed thoroughly with copious amounts of running water. If SEcure components enter the eye, rinse immediately and thoroughly with running water and have the patient examined by an ophthalmologist. Clinicians should use dental gloves.
- ③**Pulp protection**  
If the preparation approaches the pulp, apply a protective liner or base.
- ④**Flammability**  
The Primer is flammable. Keep away from open flame in use and storage.
- ⑤**Applications**  
Only use SEcure for the applications recommended in these instructions.

#### 3.2 To get the best results with SEcure

- ①**Storage conditions**  
**The kit should be stored IN A REFRIGERATOR.** High temperature and direct sunlight will shorten their shelf-life.
- ②**Avoid Contamination**  
To avoid contamination, do not mix the components with other materials. Firmly recap the containers immediately after use. After the cementing procedure, dispose of any excess of mixed materials. Once dispensed do not return materials to the containers.

***As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.***

#### 4. How to use SEcure

**Note :** The use of eugenol-based temporary materials may compromise the polymerization of SEcure.

##### ①Surface preparation

##### 1) Tooth Surface

Clean cavity/preparation, wash with water and dry with oil-free air. (A rubber dam is highly recommended to reduce the chance of contamination by saliva,

breath humidity or blood.)

Then apply the Primer to the tooth surface. After 20 seconds, spread the primer by gentle air blow to make the thin film and looks shiny surface.

#### 2) Restoration Surface

To the extent practically possible, the surface of a restoration to be bonded should be abraded using a sand-blaster or roughened by a round bur. Rinse thoroughly and dry.

Then, depending on the restorative material, the roughened and cleaned surface may be treated with a surface modifier, such as Porcelain Liner M for porcelain or V-PRIMER for precious metals, to get higher bond strength and durability.

#### ②Mixing and application of the cement

- **Important :** Remove sealing cap from the dual-barrel cartridge and express a small amount of base and catalyst until they flow equally from both orifices.
- Attach and lock a mixing tip onto the cartridge and express a small amount of mixed material to confirm good flow and mixture.
- Apply the mixture to the surfaces of the prepared tooth and/or the restoration.
- Carefully wipe the orifices and re-seal cartridge taking care to align the V on the cap with the notch on the cartridge collar. This will assure that the proper plugs fit the holes. (If the plugs are reversed, the cement may set in the syringe.)

**Note :** The mixture should be used within 30 seconds of mixing.

#### ③Cementing, setting and finishing

- Insert the restoration immediately after the application of the cement. After confirming that it is completely seated, hold it under pressure in position until the cement sets.
- To remove the excess cement, irradiate the excess with visible light for 1 - 3 seconds and immediately remove it. Or, alternatively without light, remove the excess on the halfway to setting, specifically 1.5 - 2 minutes after cementing.
- The final setting time is approximately 5 minutes in the mouth, thereafter polish the cement line or adjust occlusion according to normal practices.

**Option : Light-setting**

If the restoration is translucent like porcelain or cured composite resin, light-setting may be opted. In this case, irradiate visible light first for 1 - 3 seconds, remove the excess cement, and then irradiate the light another 20 seconds for the final setting.

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