

NEW

Brush-dip

Based on Superbond technology

A new splinting System

- Mobile teeth splinting
- Post Orthodontic splinting

No metal wire
No synthetic fibers
No adhesive
No composite

ONLY
BrushDip



Courtesy of Dr Cazier (Paris-France)



A very thin
and esthetic splint

So easy to use



Brand new powder with spherical particles for a very nice mix with the Liquid.



New plastic dispensing dish you can use at local temperature.



Enamel etching gel included in the kit.

Acid etching of enamel with the Gel



- Dispensing the Brushdip Powder In a dispensing cup.
- Dispense 4 Monomer drops and 1 Initiator drop from the syringe in an other dispensing cup.
- Soak the brush with mixed Monomer and Initiator.
- Brush this liquid onto the enamel surface to be bonded.



- Touch the powder with the Brush with a slow circular motion.
- A small ball of resin will be formed on the wet tip of brush.



- Left view: the resin ball is perfect.
- Right view: there is some excess of powder
- When applying on enamel gently tap the brush and it will wet the white resin particles and form a smooth surface on enamel.



- Apply Brushdip on the etched enamel
- Clean up the brush with gauze and repeat the procedure

Optional: you can apply the activated liquid (initiator + monomer) onto the enamel etched surface and then apply the procedure: dip the brush into the liquid and form the resin ball

- **Setting time: 5 mn**

Polishing can be achieved during the next appointment (using low speed with water)

Reloading the Brushdip splint is always possible if necessary.

The BrushDip kit contains:

- 1 Initiator 0,3 ml
- 1 Enamel etching gel 1 ml + needle tips
- 1 Monomer 3,5 ml
- 1 BrushDip Clear powder 1,5 g
- 1 Dispensing dish + Cups
- Brushes and handle
- 1 Instructions booklet
- 1 Working card