■ **Related Products of Super-Bond**
(sold separately)

- **Porcelain Liner M**
Adhesive primer for porcelain
- **V-PRIMER**
Adhesive primer for precious metal alloys
- **Super-Bond SEP**
Water soluble release agent

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■ **Accessories for Super-Bond**
(sold separately)

- | | |
|--|--|
| Dispensing Dish (Ceramic) | Brush Handle (Bent) |
| Dispensing Stand | Brush Tips (Bulk-mix) Label color: Blue |
| Dispensing Cups 40 pieces | Brush Tips (Brush-dip S) Label color: Green |
| Sponge (L・S) | Brush Tips (Brush-dip L) Label color: Pink |
| Measuring Spoon (Standard) for the standard mix ratio | Brush Tips (Brush-dip LL) Label color: Purple |
| Measuring Spoon (Small) for 75% of the standard mix ratio | Needle Tips (23G) 50 pieces and Needle Caps (Red) 2 pieces |
| Measuring Spoon (Large) for 120% of the standard mix ratio | Needle Tips (23G) 50 pieces and Needle Caps (Green) 2 pieces |
| Brush Handle (Straight) | |

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EN FR DE IT ES

INSTRUCTIONS

IMPORTANT:
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Dental Adhesive Resin Cement

**Super-Bond
Brush-dip
Starter Kit**

FOR DENTIST USE ONLY

Super-Bond Brush-dip Dental Adhesive Resin Cement Starter Kit

Made in Japan by

SUN MEDICAL CO.,LTD.

571-2, Furutaka-cho, Moriyama, Shiga, 524-0044, Japan
Phone: 81-77-582-9981 Fax: 81-77-582-9984

EC-Representative : **MITSUI CHEMICALS EUROPE GmbH**
Oststr. 10, 40211 Düsseldorf, Germany
Phone: +49 (0) 211-17332-0 Fax: +49 (0) 211-323486

2.Contents of Super-Bond Brush-dip Starter Kit

CAUTION

① Avoid contact

Avoid contact with soft tissue, skin or eyes. A rubber dam is recommended for intraoral use. Dentist should use rubber or PVC dental gloves. Contaminated skin or mucosa should be wiped off immediately with alcohol and then thoroughly rinsed with running water, otherwise symptoms such as swelling may show. If Super-Bond Brush-dip Starter Kit enters the eye, immediately rinse thoroughly with running water. The patient should be examined by an ophthalmologist. When the cured adhesive contacts soft tissue, polish the adhesive surface.

② Be careful of acidity


As the Enamel Etchant Gel is acid, avoid contact with soft tissue, skin or eyes, and do not allow patients to swallow it while applying and washing it with water.

③ Be careful of flammability

Brush-dip Initiator and Brush-dip Monomer are flammable. Do not store where they may be exposed to open flame.

④ Clean spilled Brush-dip Initiator immediately with wet towels

The Brush-dip Initiator reacts with oxygen. If absorbed by a flammable material, it may raise the temperature enough to cause smoldering. If the Brush-dip Initiator is spilled, wipe it up immediately with a **WET(not dry)** disposable towel. Then rinse the towel to kill the initiator thoroughly in running water.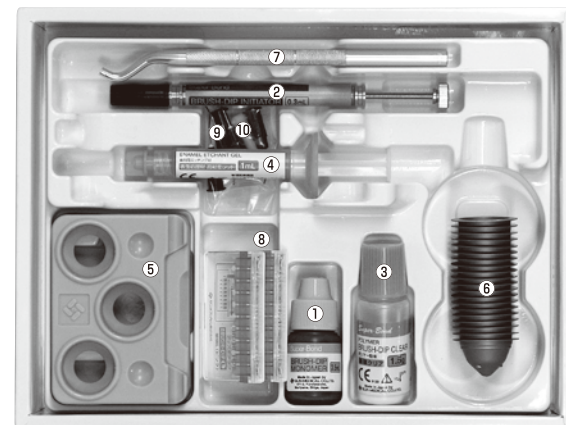

Super-Bond Brush-dip Starter Kit

① Brush-dip Monomer -----	3.5mL	⑥ Dispensing Cups -----	20
② Brush-dip Initiator -----	0.3mL	⑦ Brush Handle (Bent) -----	1
③ Polymer (Brush-dip Clear) -----	1.5g	⑧ Brush Tips (Brush-dip L) Label color: Pink -----	20
④ Enamel Etchant Gel -----	1mL	⑨ Needle Tips (23G) -----	5
⑤ Dispensing Stand -----	1	⑩ Needle Cap (Red) -----	1

■ Attached documents:
Instructions / Pictorial Instruction Card

1.What is Super-Bond Brush-dip Starter Kit?

Super-Bond Brush-dip Starter Kit is a self-cure dental adhesive resin cement based on MMA. It contains a high performance bonding monomer, "4-META", and an initiator, "TBB". It shows excellent bond strength to tooth (enamel and dentin), metal*1, porcelain*2 and dental resins.

For three decades it has been used extensively in a wide range of dental applications. During this long period it has earned an excellent reputation for pulpal safety.

Super-Bond Brush-dip Starter Kit is widely known for its formation of a sound hybrid layer (resin impregnated layer) in both enamel and dentin. This layer reinforces the tooth surface against recurrent caries and prevents the post- operative hypersensitivity.

*1 Use V-PRIMER concurrently for precious metal alloys.

*2 Use Porcelain Liner M concurrently for porcelain.

3.Precautions

Read all instructions thoroughly before use.

3-1 Safety

Please keep the following precautions for safe use.

(Regarding Brush-dip Initiator, read 3-2 additionally.)

①Applications

Use Super-Bond Brush-dip Starter Kit only for the applications recommended in this publication.

②Past history of sensitivity

Super-Bond Brush-dip Starter Kit should not be used by clinicians or on patients who are methacrylic monomer- sensitive.

③Symptomatic irritation

Cease using Super-Bond Brush-dip Starter Kit immediately, if signs of irritation such as rashes appear, and see a physician.

④Avoid contact

Avoid contact with soft tissue, skin or eyes. A rubber dam is recommended for intraoral use. Dentist should use rubber or PVC dental gloves. Contaminated skin or mucosa should be wiped off immediately with alcohol and then thoroughly rinsed with running water, otherwise symptoms such as swelling may appear. If Super-Bond Brush-dip Starter Kit enters the eye, immediately rinse thoroughly with running water. The patient should be examined by an ophthalmologist. When the cured adhesive contacts soft tissue, polish the adhesive surface.

⑤Be careful of acidity

As the Enamel Etchant Gel is acid, avoid contact with soft tissue, skin or eyes, and do not allow patients swallow it during application or rinsing.

⑥Pulp protection

If the preparation approaches the pulp, apply a protective base.

⑦Give care to flammability

Brush-dip Initiator and Brush-dip Monomer are flammable. Do not store where they may be exposed to open flame.

3-2 Precautions on Brush-dip Initiator

Brush-dip Initiator reacts with air and water to generate heat and lose activity. Please abide by the following.

①Storage conditions

Avoid high temperature, high humidity and direct sunlight.

*After a long storage, the first drop of the Brush-dip Initiator may be inactive, though the rest of the material remains active.

*The syringe is made of glass, therefore it must be handled with care to prevent shock, dropping, and other physical damage.

②Cap closure

The cap simply slides on and off. Recap the syringe immediately after each use. Air (oxygen and humidity) deactivates the Brush-dip Initiator. Do not leave the cap off during the bonding procedure.

③Screwing

If the Brush-dip Initiator does not come out of the syringe because of the tight screw, do not try too hard to turn it. The content may splash as the syringe breaks.

④After use

Unscrew the male-screw two turns counter-clockwise after each use to relieve pressure on the Brush-dip Initiator. (Pressure buildup can cause leakage of the Brush-dip Initiator or a crack of the syringe.)

⑤Clean spilled Brush-dip Initiator immediately with wet towels

The Brush-dip Initiator reacts with oxygen. If absorbed by a flammable material, it may raise the temperature enough to cause smoldering. If the Brush-dip Initiator is spilled, wipe it up immediately with a **WET(not dry)** disposable towel. Then rinse the towel to kill the Brush-dip Initiator thoroughly in running water.

⑥Cleaning of the tip of the syringe

Wipe the tip of the syringe with a dry gauze after each use to prevent residue buildup. Then rinse the gauze with water to kill any remaining activity. Buildup of the residue may prevent the cap from seating properly.

3-3 Storage

Please take the following precautions to maintain the quality.

①Storage conditions

Store the Brush-dip Initiator, Brush-dip Monomer, Polymer and Enamel Etchant Gel in a cool and dark location. High temperature, high humidity and direct sunlight will shorten their shelf-life.

②Volatility

Brush-dip Monomer is highly volatile. Recap the bottle immediately.

③Contamination

Do not mix the bottle caps.

④Dispensing Cups and Brush Tips disposal

Both Dispensing Cups and Brush Tips are for one-time use. Dispose of them after use.

3-4 To Get the Best Results with Super-Bond Brush-dip Starter Kit

①Create and maintain a clean surface

Oil, blood, saliva and biofilm will lower the bond strength. Clean the tooth and prosthesis thoroughly before cementing. After cleaning, take care to avoid re-contamination.

②Dry the surfaces and prevent moisture contamination

After cleaning, dry the surface adequately. A rubber dam is highly recommended, as it will reduce the chance of contamination by saliva, humid breath or blood.

③Avoid eugenol-containing bases and cements

Eugenol is a polymerization inhibitor. Therefore, eugenol-contained bases and cements should not be used with resin cements. To avoid cross-contamination, reserve a mixing dish exclusively for Super-Bond Brush-dip Starter Kit. Do not use the same mixing dish for other adhesives.

④Time constraints

Super-Bond Brush-dip Starter Kit's working and setting times are very different from those of traditional cements. Follow the instructions carefully to get the best results.

⑤Do not re-use Polymer

After using Super-Bond Brush-dip Starter Kit in the Brush-dip technique, dispose of any excess Polymer left in the mixing dish. Do not return it into the container, as it has become contaminated with the Brush-dip Monomer.

⑥Prosthesis Design

To avoid stress concentration which encourages debonding regardless of the actual bond strength, design a prosthesis, such as wings of a bonded bridge, without thin unsupported area, which may flex during mastication.


As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.

4. How to use Super-Bond Brush-dip Starter Kit

Surface Preparation


It is essential that all surfaces to be bonded with Super-Bond Brush-dip Starter Kit should be properly prepared. Preparation varies depending on the nature of the materials.

Tooth Surface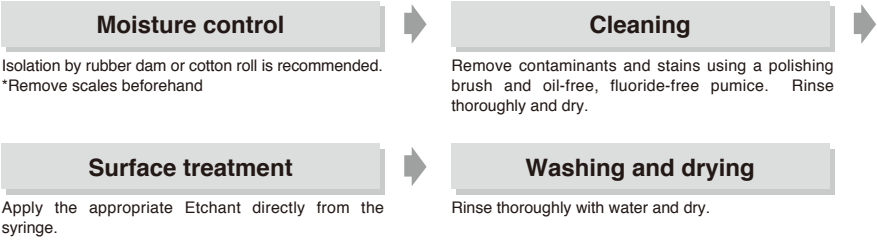


Treatment time	
Etchant	Enamel Etchant Gel
Enamel	30-60 sec

Porcelain Surface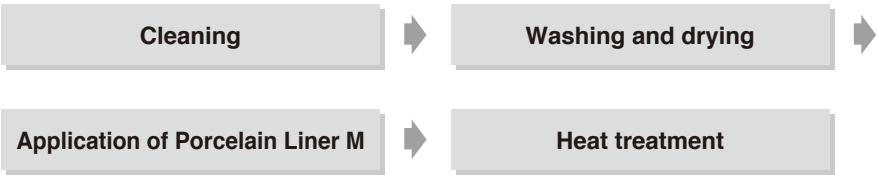

Metal Surface

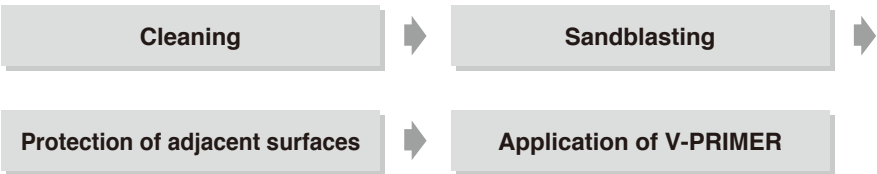Operation Steps

BRUSH-DIP TECHNIQUE

Dispensing the Polymer

Dispense the appropriate amount of Polymer powder into a Dispensing Cup.

Preparation of the Activated Liquid

Hold the Brush-dip Monomer bottle vertical and dispense the appropriate number of drops into another Dispensing Cup.


Hold the Brush-dip Initiator syringe vertical, and turn the screw to dispense the proper number of drops into the Brush-dip Monomer. Stir lightly with a brush. This mixture is called "activated liquid"


Application of the Activated Liquid

Brush the liquid onto the surface to be bonded.
*The activated liquid decomposes gradually and loses activity. Use it within 5 min. after preparation.

Brush-dip Procedure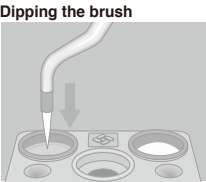

Dip the Brush Tip into the activated liquid.
*When you repeat the procedure, clean up the brush with gauze before you dip it.

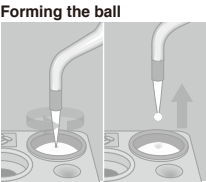Touch the brush to the Polymer powder in the Dispensing Cup. A small ball of powder will be picked up on the wet tip of the brush.

Applying the ball
Brush the powder ball onto the pre-wet surface being bonded. As soon as it touches the surface, the powder will spread out to create a creamy, homogeneous layer. If necessary, repeat the procedure until the entire surface is covered with the cement.

Seating the Restoration

Insert the restoration immediately. After confirming that it is completely seated, hold in position until the cement sets.

Post Treatment

Remove the excess cement.
To facilitate this, protect any adjacent surfaces you will not be bonding, and remove the excess resin before it sets.

Mixing ratio

Brush-dip Monomer	Brush-dip Initiator
4 drops	1 drop
8 drops	2 drops

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